

## Chloraprep® surgical applicators

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v

Licensed, single use, sterile, ANTT™, latex free, skin antiseptics system

### Pinch



Hold the applicator as shown, being careful not to touch the sponge.

Pinch the lever to release the solution. You will hear a 'pop' as the ampoule breaks.

### Apply



Starting at the incision site, gently press the applicator against the skin until the solution soaks the sponge.

Apply using repeated up and down, back and forth strokes for at least 30 seconds, before working outwards to the periphery.

### Dry



Leave the area to air **dry completely** before applying sterile drapes. Do not blot or wipe away. Discard the applicator after a single use.

**Important safety point.**

Do not drape or use ignition source until the solution has completely dried.

#### 3ml applicator

Coverage area  
15 cm x 15 cm



#### 10.5ml applicator

Coverage area  
25 cm x 30 cm



#### 26ml applicator

Coverage area  
50 cm x 50 cm



**For procedures such as:**

Surgical site antiseptics  
Implantable device placements  
Prosthetic device placements  
Femoral graft sites

The 26ml applicator contains two swabs. Clean umbilicus with swabs when applicable (moisten swabs by pressing against soaked sponge).

**For customer services and all other enquiries:** Please telephone: 0800 043 7546

Or visit: [www.chloraprep.co.uk](http://www.chloraprep.co.uk)

CHL065c Sept 2008

**Chloraprep®**  
Patient Preoperative Skin Preparation  
2% Chlorhexidine Gluconate (CHG) w/v  
& 70% Isopropyl Alcohol (IPA) v/v