

# ChloroPrep FREPP 1.5ml cutaneous solution

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v

Licensed, single use, sterile, ANTT,<sup>™</sup> latex free, skin antiseptic system

## Pinch

Hold the applicator as shown, being careful not to touch the sponge.  
Pinch the wings together. You will hear a 'pop' as the ampoule breaks.  
Hold horizontally to aid even spread of the solution through the sponge.



## Apply

Gently press the applicator against the skin until the solution soaks the sponge. Apply using repeated up and down, back and forth strokes for at least 30 seconds.



## Dry

Leave the area to air **dry completely** before proceeding.  
Do not blot or wipe away.

Discard the applicator after a single use.



## FREPP 1.5ml



NHS list price per applicator:  
£0.55 (ex. VAT and delivery)

NHS supply chain  
order code: MRB303

**Coverage area:**  
10 cm x 13 cm

**For procedures such as:**

- Blood culture collections
- Peripheral cannulation
- AV fistula and 'portacath' access
- Simple biopsies
- Minor procedures

### Prescribing Information

ChloroPrep® (PL31760/0002) & ChloroPrep with Tint (PL31760-0001) 2% chlorhexidine gluconate w/v / 70% isopropyl alcohol v/v cutaneous solution. **Indication:** Disinfection of skin prior to invasive medical procedures. **Dosage & administration:** ChloroPrep – 0.67ml, 1.5ml, 3ml, 10.5ml, 26ml; ChloroPrep with Tint – 3ml, 10.5ml, 26ml. Volume dependent on invasive procedure being undertaken. Applicator squeezed to break ampoule and release antiseptic solution onto sponge. Solution applied by gently pressing sponge against skin and moving back and forth for 30 seconds. The area covered should be allowed to air dry. **Side effects, precautions & contra-indications:** Very rarely allergic or skin reactions reported with chlorhexidine, isopropyl alcohol and Sunset Yellow. Contra-indicated for patients with known hypersensitivity to these constituents. For external use only on intact skin. Avoid contact with

eyes, mucous membranes, middle ear and neural tissue. Should not be used in children under 2 months of age. Solution is flammable. Do not use with ignition sources until dry, do not allow to pool, and remove soaked materials before use. Over-vigorous use on fragile or sensitive skin or repeated use may lead to local skin reactions. At the first sign of local skin reaction, application should be stopped. **Per applicator costs (ex VAT)** ChloroPrep – 0.67ml (SEPP) - 30p; 1.5ml (FREPP) - 55p; 3ml - 85p; 10.5ml - £2.92; 26ml - £6.50. ChloroPrep with Tint – 3ml - 89p; 10.5ml - £3.07; 26ml - £6.83. **Legal category:** GSL. **Marketing Authorisation Holder:** CareFusion UK 244 Ltd, 43 London Road, Reigate, Surrey RH2 9PW, UK. **Date of preparation:** July 2010.

CHL116a Date of preparation: August 2010