

Chloraprep® FREPP 1.5ml cutaneous solution

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v

Licensed, single use, sterile, ANTT™, latex free, skin antiseptis system

Pinch



Hold the applicator as shown, being careful not to touch the sponge. Pinch the wings together. You will hear a 'pop' as the ampoule breaks. Hold horizontal to aid even spread of the solution through the sponge.

Apply



Gently press the applicator against the skin and apply the antiseptic using repeated up and down, back and forth strokes for about 30 seconds.

Dry



Leave for approximately 30 seconds, allowing the area to air **dry completely** before proceeding.

Discard the applicator after a single use.

FREPP applicator

Coverage area
10 cm x 13 cm



For procedures such as:

- Blood culture collections
- Peripheral canulation
- AV fistula and 'portacath' access
- Simple biopsies
- Minor procedures

For customer services and all other enquiries: Please telephone: 0800 043 7546

Or visit: www.chloraprep.co.uk

CHL065d Sept 2008

Chloraprep®
Patient Preoperative Skin Preparation
2% Chlorhexidine Gluconate (CHG) w/v
& 70% Isopropyl Alcohol (IPA) v/v